

IMPORTANT - PLEASE READ FIRST

Timing: No food for 1-hour prior and no water 30-minutes prior to sample collection.

Multiple Pets: If collecting samples for more than one pet, complete one pet before starting the next.

STEP 1

Wash your hands, and if you have a friend available to help hold or swaddle your cat. Remove the swab from packaging, being careful not to touch anything with the tip.




STEP 2

Insert the tip of the swab between your cat's cheek and gum. Press gently to the inside of the cheek, RUB & ROLL against cheek while **avoiding the gums**.

STEP 3

Snap and remove the excess stick at the notch and place the swab (head first) into the tube and close to avoid spilling any liquid.

Repeat Step 2 & 3 for the second swab.

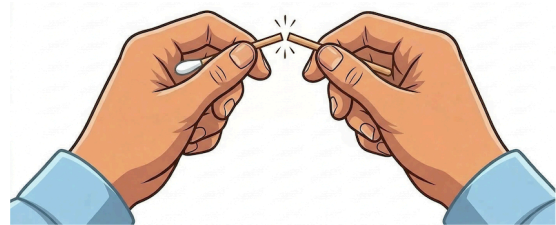

STEP 4

With both swabs inside the tube and cap secured, lightly shake the tube to ensure swabs are coated in the liquid.

Ensure the label has the pet's name clearly written.