

## IMPORTANT - PLEASE READ FIRST

**Timing:** No food for 1-hour prior and no water 30-minutes prior to sample collection.

**Multiple Pets:** If collecting samples for more than one pet, complete one pet before starting the next.

### STEP 1

Wash your hands, and if you have a friend available, they can help hold your dog. Remove the swab from packaging, being careful not to touch anything with the tip.




### STEP 2

Insert the tip of the swab between your dog's cheek and gum. Press gently to the inside of the cheek, RUB & ROLL against cheek while **avoiding the gums**.

### STEP 3

Snap and remove the excess stick at the notch and place the swab (head first) into the tube and close to avoid spilling any liquid.

**Repeat Step 2 & 3** for the second swab.

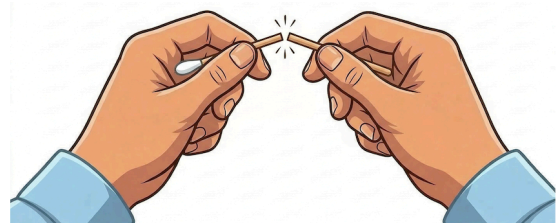

### STEP 4

With both swabs inside the tube and cap secured, lightly shake the tube to ensure swabs are coated in the liquid.

Ensure the label has the pet's name clearly written.